

Tube-Evac™

Not made with natural latex rubber. DEHP free.

More information is available www.tubeevac.com.



INTENDED USE:

The Tube-Evac™ is a manual hand-held device intended to clear or strip fluid and debris from flexible medical drainage tubes (such as surgical drains or suction tubing). It is designed for use with tubing sizes ranging from approximately 7–10 Fr (REF: TE-0710) and 15–19 Fr (REF: TE-1519). The device assists in restoring flow and maintaining drain patency by manually compressing and sliding along the tube exterior to remove clots or accumulated material.

WARNINGS / CAUTIONS:

- Tube-Evac™ does not contact patient tissue or sterile fields and is used externally-only on tubing.
- Do not use Tube-Evac™ with tubes which are larger or smaller than the specified Fr range, as integrity of the tubes may be compromised.
- Do not use Tube-Evac™ with drains that have been compromised.
- Avoid placing Tube-Evac™ near any wound, the Tube-Evac™ is a non-sterile product.
- Single patient use in order to prevent cross contamination between patients.




CLEANING INSTRUCTIONS:

- Single patient use.
- May be hand washed with mild detergent in warm water.




INSTRUCTIONS FOR USE:

Note: Confirm the applicable drain tube size prior to opening the Tube-Evac™ packaging. Opened devices may not be returned or refunded.

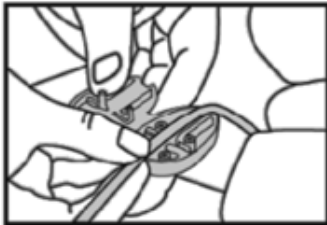

1. Place the tubing in the groove of the Tube-Evac close to where it exits the skin.

2. With your other hand, hold the tubing firmly between the skin and the Tube-Evac.

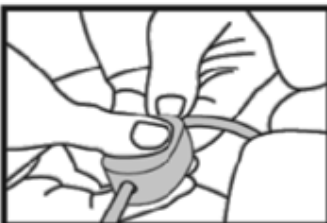

3. Firmly squeeze the Tube-Evac and pull towards the collection bulb.

4. Advance other hand as necessary to hold the tubing.

Please contact Tube-Evac with any questions or comments or to report any issue with the use of our product.



HOHT, LLC dba Tube-Evac

www.tubeevac.com

83 Wooster Heights Rd

Ste 125

Danbury, CT 06810

©Tube-Evac 2025



5. The Tube-Evac easily rolls down the tubing while applying a firm squeeze to evacuate small fibrinous material and or clots to maintain a functioning drain.

STORAGE AND DISPOSAL:

No special requirements.

ENVIRONMENTAL:

Suitable usage for usage in the hospital, home, and mobile unit(s).



TubeEvac.com

Please contact Tube-Evac with any questions or comments or to report any issue with the use of our product.



HOHT, LLC dba Tube-Evac

www.tubeevac.com

83 Wooster Heights Rd

Ste 125

Danbury, CT 06810

©Tube-Evac 2025